

Using the My-Cardia App for IOS

1. Download the free My-Cardia app from the App Store and open it.
2. Enter your First/Last Name and Date of Birth and click Register.*
3. Answer "OK" to allow App to use your microphone.
4. When you are ready to transmit ECG tap the picture of the MyCardia AT device.
5. Place the MyCardia AT device 1 inch below the the bottom of your phone with the red button to the right as shown in the app.
6. Press the red button on the MyCardia AT to start playing your recording(s).
7. When the recording has completely finished tap the message on your phone to transmit it.**

International use only.

* You must use the same name and birth date that your facility entered when you were enrolled.

**** Do not forget this step or your ECG will not be transmitted!**

